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Dedicated to  
the Benefits of Breastmilk

**Pigeon developed the nipple and bottle for the sake of babies with cleft lips and palates. Use under the guidance of medical specialist or registered nurse. Use with formula or expressed breast milk.** \* Once your baby can suck skillfully, try feeding him or her using a standard-type bottle from Pigeon.

### Nipple features

**Regular size** (Marked with an “R” on the nipple flange)

- The tip of the Regular size nipple is larger than that of a standard type nipple in order to prevent the leakage of air or milk from fissures in babies with cleft lips or cleft palates. In addition, in cases where sucking strength is weak, a slightly larger milk-flow opening makes more milk than usual flow from the nipple in order to evacuate the milk smoothly.

- The tip of the Small size nipple is smaller than that of the Regular size nipple, and the volume of milk that flows from the nipple is also less.

If the nipple used is not suitable for your baby's condition, the correct amount of milk may not be obtained per sucking session (too much or too little), and this can cause problems for the baby, such as choking or difficulty drinking. Select from the two nipple sizes based on your baby's condition and growth.

### Names of parts and their functions

#### Milk-flow opening

The milk-flow opening for the Regular size nipple is a slightly larger cross-cut opening, allowing milk to flow smoothly even when sucking strength is weak. The milk-flow opening for the Small size nipple is a cross-cut opening slightly smaller than that of the Regular size nipple. Use the Small size when the volume of milk flowing from the Regular size nipple is too large.

#### Nipple

You can now select from two sizes of nipple, made from a soft silicone rubber, to suit your baby's condition.

- The gradation marks on the nursing bottle are to aid in preparing formula. When more precision is required, use a measuring cup or measuring spoon.

**EN** Be sure to read before use.

\* Keep this User's Manual in a safe place.

Product Name: Nursing bottle (plastic)

| Parts                          | Material        | Heat resistance | Sterilization methods |
|--------------------------------|-----------------|-----------------|-----------------------|
| Hood                           | Polypropylene   | 120°C           | Boiling               |
| Cap                            |                 | 120°C           |                       |
| Inner lid                      |                 | 120°C           |                       |
| Backflow prevention valve (x2) |                 | 120°C           |                       |
| Bottle                         | Silicone rubber | 110°C           | Microwave             |
| Nipple (Regular size)          |                 | 120°C           |                       |
| Nipple (Small size)            |                 | 120°C           | Chemicals             |

#### Shape of milk-flow opening

Regular size: Cross cut (regular)

Small size: Cross cut (small)

Bottle volume: 240ml (highest gradation volume)

#### Handling Cautions

- After use, wash the bottle and nipple thoroughly using the dedicated brushes and then sterilize.
- Do not leave a nipple in direct sunlight or heat, or leave in disinfectant (“sterilizing solution”) for longer than recommended, as this may weaken the nipple.
- Use a sponge brush to wash. Using a hard brush for cleaning will cause damage.
- This nursing bottle and nipples cannot be used together with other products.

### Concerning compatibility

Backflow prevention valves without the Pigeon mark (illus. ①) cannot be attached to transparent silicone rubber nipples. Be sure to use a backflow prevention valve with the Pigeon mark (illus. ②).

Visit our breastfeeding support website for new mother and father!

**A** Breastfeeding Advisor™

**EN** ba.pigeon.com.sg

**RU** ba.pigeon.ru

**CS** ba.pigeon.cn

PIGEON

PIGEON CORPORATION

4-4 Nihonbashi Hisamatsu-cho,

Chuo-ku, Tokyo 103-8480 JAPAN

www.pigeon.com

MADE IN CHINA

CODE NO. 00906

### How to feed

- Hold the bottle with the thicker side of the nipple and ventilation valve facing upward so that baby can press on the bottom (thinner) side of the nipple to drink milk. \* The side of the nipple with the ventilation valve is thicker, and slightly flatter, so even where there is a fissure, the nipple adheres easily.

- Baby's sucking motion allows milk to collect inside the nipple and then flow out so that he or she can drink, but if sucking strength is weak or your baby has little strength to press on the nipple, milk may not collect inside the nipple. If this happens, squeeze the bottle gently to fill the nipple with milk.

- The upper part of the bottle is soft so that it is easy to squeeze. Squeezing this area allows you to comfortably adjust your squeezing strength.

- If you squeeze too hard on the bottle, milk may flow too heavily from the tip of the nipple, and this may cause your baby to choke. Regulate your squeezing strength while guarding against choking.



- Cool the milk to body temperature before feeding.

- Hold your baby in your arms while feeding.

- While feeding, check periodically to make sure that the ventilation valve is not stuck.

- Feed as soon as possible after preparing formula. Discard any formula not used within two hours of preparation.

The thicker side of the nipple with the ventilation valve should be up.

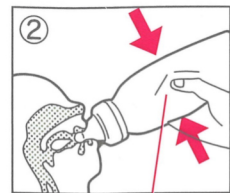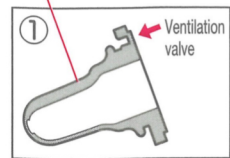Easy to squeeze

### When to replace nipples

- If your baby grows comfortable with one certain nipple, he or she may object if you change to a new one. Rotate 2 or more nipples for feeding. If you use the same nipple for an extended period of time, it will lose elasticity and tear easily, which could cause unexpected accidents.
- If the surface of the nipple becomes sticky or the tip begins to bulge, the rubber has begun to deteriorate. Do not continue use of this nipple.

- When to replace nipples / When nipples have been sterilized by boiling for an accumulated time of 24 hours. / When nipples have been sterilized by steaming using an autoclave for an accumulated time of 20 hours.
- Without regard to how long nipples have been sterilized or how many times they have been used, discard them at the first signs of abnormality, including tears, cracks or deterioration, or after two months since the nipple was first used.

- “Accumulated time” does not include the time when the temperature is being increased or decreased.

#### Warning against burns

**Warning:** Be aware of the following, as they can lead to burns.

- When preparing formula, the bottle also becomes very hot. Be careful not to burn yourself.
- Do not prepare formula next to your baby. Doing so may lead to burns.
- Only begin feeding after first testing the temperature of the milk or other liquid. Not doing so may lead to burns.
- When sterilizing by boiling, the bottle will become very hot. Be careful not to burn yourself.
- If you shake the bottle up and down when preparing formula, the contents may spurt out, and this may lead to burns.

- Nursing bottles are to be used to feed your baby milk or other liquids. Do not use them for any other purposes.
- To protect the ventilation valve and milk-flow opening (cross cut), a white powder food additive has been applied to the nipple. Be sure to wash thoroughly before using for the first time.

For your child's safety and health

#### WARNING!


- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperature before feeding.
- Keep all Components not in use out of the reach of children.
- Inspect before each use and pull the nipple in all directions.

### Cleaning

 Always keep equipment as hygienic as possible.

- Before use, including before using for the first time, be sure to wash and sterilize all items.
- After use, immediately rinse in warm water and wash using a Pigeon Liquid Cleanser.
- Wash the nipple thoroughly, taking particular care with the ventilation valve. If the valve becomes blocked, it may cause the nipple to collapse or be sucked into the bottle.

- When washing the nursing bottle, do not use abrasive or metallic brushes or abrasive cleansers, as doing so may scratch the bottle. Such damage may lead to bottle breakage.

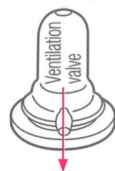### Formula preparation

See the milk powder package for details on how to prepare formula.

- Be sure to use water that has been boiled and then cooled to just over 70°C to prepare formula. If water has been boiled and then left for over 30 minutes and/or cooled to below 70°C, do not use it under any circumstances. In winter, the bottle will be cold, so if you add water for formula that is about 70°C, the temperature of the hot water will drop. Before preparing formula, warm the bottle by first rinsing it with warm water. Discard that water and then add water at 70°C.

- The directions for preparing formula vary depending on the milk powder manufacturer. Follow the instructions for making formula from each milk powder manufacturer.

- Attach the nipple to the bottle. Use a clean cotton towel or pot holder to hold the bottle, in order to prevent burns, and swirl the bottle to thoroughly mix the contents.

- If you shake an upright bottle up and down, the contents may gush out, and this can cause burns.

- If you shake an upright bottle after attaching the nipple, the hot water will cause the pressure inside the bottle to rise, and this may cause the contents to leak or gush out.

- After preparing the formula, attach the nipple and cool the contents to a temperature appropriate for feeding, either by leaving the bottle in a bowl filled with water or by holding the bottle under cold running water.

- Keep the bottom part of the bottle under running water to make sure that the nipple does not come into contact with the water.

- If you shake the bottle after attaching the hood, the hot water will cause the pressure inside the bottle to rise, and this may cause the contents to leak or gush out.

- The milk-flow opening is a cross-cut opening, so even if you tip the bottle over, milk will not leak out. After cooling the bottle thoroughly, check the outside temperature of the bottle.

- Avoid using a microwave oven to heat water to make formula or to heat prepared formula.

- Avoid preparing formula next to your baby, as this is dangerous.

### Sterilization methods

Select a sterilization method based on the properties of microbes and the environment of use.

#### Steam Sterilization (Autoclave)

In accordance with the autoclave handling directions, sterilize items either for 15 minutes at 121-124°C or for 3 minutes at 135°C. \* For this product, only the nipple can be sterilized by autoclaving. \* The color of the silicone rubber may change due to sterilization. We recommend that you switch to a new nipple as soon as possible if the color changes.

#### Chemical Sterilization

Use only a chemical disinfectant meant for use with nursing bottles and nipples. For details on use, see the instructions for the disinfectant. Always comply with the warnings on the disinfectant package. \* The nipple may absorb water and turn a white color, but this is not a problem for use.

#### Boiling Sterilization

Select a boiling time based on the characteristics of the microbes you want to control, the number of babies who might be using the product and the environment in which it will be used. \* If plastic parts or nipples come into contact with the bottom or sides of the container, their shapes may deform, so take care. \* Be sure to add plenty of water so that the container does not boil dry. \* Stay next to the container while you are sterilizing items by boiling.

#### Microwave Sterilization

Be sure to use specialized sterilization equipment. Nursing bottles and other parts cannot be placed directly into the microwave for sterilization. \* Follow the directions for the sterilization equipment.

#### Be sure to observe the following.

Do not use any sterilization methods other than autoclaving, chemical solutions, boiling or microwaving to sterilize rubber parts, as doing so may cause the parts to deteriorate, lose strength, or tear.

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